




Hands-Free Smoke Capture

Instructions for Use:

1. Visually inspect the package for breaches of packaging integrity prior to use.
2. Before placement, prepare skin and apply drape per manufacturer's procedure.
3. Remove sterile Flamingo Funnel™ product from the pouch.
4. Remove release paper on the bottom of the Flamingo Funnel to expose the adhesive layer.
5. Affix Flamingo Funnel to the drape or skin, within 0.5 inches to 1.5 inches of intended line of incision, on the side that allows the tubing to remain straight when connected to the smoke evacuator.
6. Remove tape from the tubing and hand it off from the operating table.
7. Place outer surgical drape over the Flamingo Funnel. Avoid covering the open cell foam.
8. Attach tubing to suction source. Use extension tubing if required.
9. Activate suction device and set to 100% continuous power. Adjust as needed for optimal smoke removal.
10. Initiate procedure.
11. After procedure, remove Flamingo Funnel and dispose of as hazardous material.

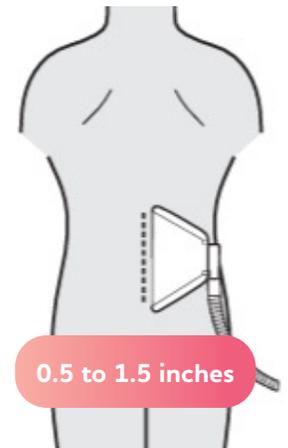

Warnings

- Single patient use only. Reuse may cause an infection.
- Do not use if sterile package is opened or damaged. Do not re-sterilize.
- Do not use if device is damaged.
- Do not use if the device has been dropped, as the device may be broken even if a break cannot be seen.
- Do not use the device after the expiration date on the label.
- Visually inspect and do not use if there is any visible particulate matter.
- Ensure use of aseptic techniques throughout the procedure.

Contraindications:

- No known contraindications

